

*Vibra*LITE can help IN A NOISY ENVIRONMENT



High Noise Levels often make it impossible to hear an audible alarm. High Noise Levels often occur while enjoying Recreation or when in a Job Placement. Regardless of the noise you need to know when to perform a task, time a procedure, take a break or lunch, take medication etc. Help is available by using the “*Vibra*LITE” vibration alarm watch with auto repeat countdown timer. Vibrations on your wrist alert and remind you as often as you need them even when an audible alarm cannot be heard.

Set the Countdown Timer for the required amount of time (i.e. every 20 minutes to perform a task or every 3 hours to take medication etc.) The Auto-Repeat Feature on the Countdown Timer on “*Vibra*LITE 3” and “*Vibra*LITE 8” will automatically reset and countdown for the preset time each time it reaches zero. These “interval” reminders are preset once and continue to repeat until you turn the countdown OFF with a push of a button. On “*Vibra*LITE 8”, it counts each time the countdown reaches Zero, so you can know exactly how many times you have performed the task. The Countdown Timer on “*Vibra*LITE 3” and on “*Vibra*LITE 8” will provide you with a short one second reminder at 10 minutes and again at 5 minutes before zero. This little silent reminder is very often helpful. On “*Vibra*LITE 8” you can set this reminder feature for the one second reminder at 10 minutes and again at 5 minutes before zero to ON or OFF as you prefer.

On “*Vibra*LITE 8” you can preset up to 8 alarms for various daily reminders. If you need an early wake up call, “*Vibra*LITE” is the perfect solution if you do not want to wake a sleeping partner or other household members. Set one alarm for the alarm clock time and one or more alarms for the snooze feature. Other alarms can be set for other daily reminders (i.e. appointments, being on time, keeping on schedule.) These alarms will alert you each day at your preset times. Alarms can be turned ON or OFF easily in the Option Mode when wanted. On “*Vibra*LITE 3” you can preset 2 alarms for variable reminders.

The hourly alert feature reminds you every hour on the hour. It is a short 1 second Vibration.

Disclaimer: In no event shall Global Assistive Devices, Inc. be held liable for any special, incidental, indirect or consequential damages whatsoever including, but not limited to, damages for loss of property, personal injury, loss of privacy, failure to meet any duty, negligence, and any other loss whatsoever, however caused, even if Global has been advised of the possibility of such damages. This disclaimer of liability applies to any damages or injury, including but not limited to those caused by any failure of performance of the product. Responsibility for selection, safety and suitability of any product rests with the consumer. Circumstances or consequences arising out of or in any way related to the use of or inability to use our products is the responsibility of the consumer. Global Assistive Devices, Inc. cannot be held liable in any way.

*Vibra*LITE can help IN A NOISY ENVIRONMENT

“*Vibra*LITE 8” has 3 Time Zones that can be set. One can be set for current location, one for home location, and one for traveling to location or GMT.

“*Vibra*LITE 3” has two Time Zones that can be set.

Disclaimer: In no event shall Global Assistive Devices, Inc. be held liable for any special, incidental, indirect or consequential damages whatsoever including, but not limited to, damages for loss of property, personal injury, loss of privacy, failure to meet any duty, negligence, and any other loss whatsoever, however caused, even if Global has been advised of the possibility of such damages. This disclaimer of liability applies to any damages or injury, including but not limited to those caused by any failure of performance of the product. Responsibility for selection, safety and suitability of any product rests with the consumer. Circumstances or consequences arising out of or in any way related to the use of or inability to use our products is the responsibility of the consumer. Global Assistive Devices, Inc. cannot be held liable in any way.